forma foam


N

PU-Wundauflagen, non adhesive
PU wound dressings, non-adhesive
Pansements en polyuréthane, non adhésifs
medicazioni per ferite in PU, non adesive

forma foam

Soft

PU-Wundauflagen, Silikon
PU wound dressings, silicone
Pansements en polyuréthane, silicone
medicazioni per ferite in PU, silicone

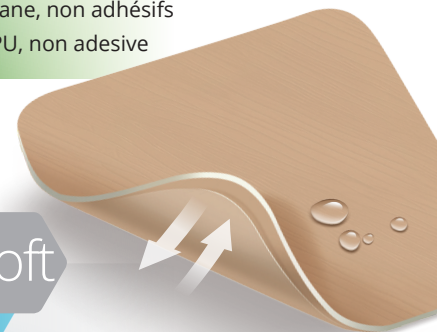STERILE | R

Rev. 2016-04-01

forma
foam 

non-adhesive,
non adhésifs,
non adhesive 5mm

forma
foam 

Silikon,
silicone 2mm

mediset

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Instructions for use

Product description:

FormaFoam is a sterile, hydrophilic polyurethane foam dressing to treat moderately to severely exuding wounds. The membrane absorbs liquid and reduces it by means of vaporisation. FormaFoam is permeable to air and water vapour and does not adhere to the wound, relieving pain when the dressing is changed. This ensures that newly formed tissue is not damaged. FormaFoam can remain on the wound for 1 to 7 days.

Indications:

- skin lesions
- operation wounds healing by secondary intention
- pressure ulcers (decubitus)
- ulcers associated with diabetic foot
- venous and arterial leg ulcers, etc.
- acute wounds
- first/second-degree burns
- split-skin graft sites and donor sites

Note:

In case of any questions or uncertainty, always seek the advice of a doctor or nursing staff. In case of an infection, the wound must be cleaned regularly and examined by a doctor.

Packaging and sterility:

Individually packaged and gamma sterilised.

Dressing a wound:

Clean the wound and remove any dead tissue following standard wound management steps or a doctor's instructions. Choose a dressing with a membrane 1.5 cm larger than the wound.

Changing dressings:

Do not leave the dressing on the wound for more than 7 days. The FormaFoam dressing must be changed before exudate spreads to the wound periphery, or earlier if wound liquids approach the edge of the dressing.

Store in a cool, dry place.

Only sterile if the sterile packaging is undamaged.

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